

APPLICATION

1. Unlock straps on brace.
2. Spread hinges apart and lay brace out flat. (Fig. 1)
3. Positioning knee centered between the hinges on brace, adjust strap lengths as needed for proper circumference. (Fig. 2)
4. Remove gator lock at ends of straps and trim. Reapply gator lock. (Fig. 3)
5. Push hinge arm adjustment buttons and slide hinge arms as needed for proper leg length. (Fig. 4)
6. Position hinge bars medially and laterally, centering the axis of the hinge with the knee center.
7. Wrap straps above and below knee and secure.
8. Wrap and secure 2 remaining straps.

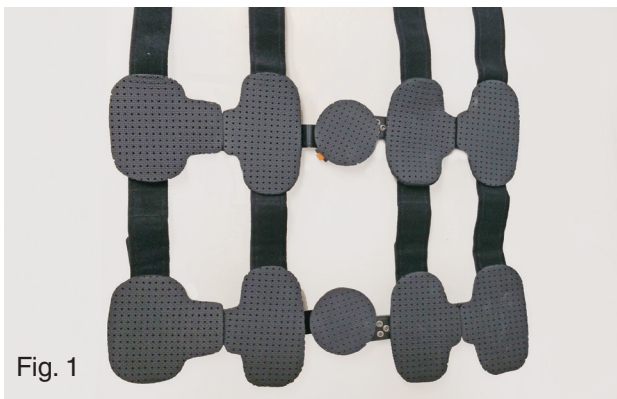


Fig. 1



Fig. 2



Fig. 3



Fig. 4



Phone: (727) 573-1595 or (800) 627-1595
 Fax: (727) 573-1886 or (800) 545-7938
 12221 33rd Street N, St Petersburg, FL 33716
 www.rcai.com

06/16 Literature may not reflect current manufacturer specifications.




Universal ROM Knee 95ROM-KO



95ROM-KO

The new **Universal ROM Knee Brace** by RCAI is indicated for controlled range of motion following ACL, PCL, MCL, LCL injuries, meniscal repairs, patella tendon repairs, condylar fractures, sprains, strains, instability of the knee and total knee replacement. The Universal ROM Knee Brace features easy to use push buttons to lengthen hinge arms, and push tabs to set flexion/extension settings and static lock. Foam padding assures patient comfort and discourages migration on the limb. One size fits all and reduces inventory needs.

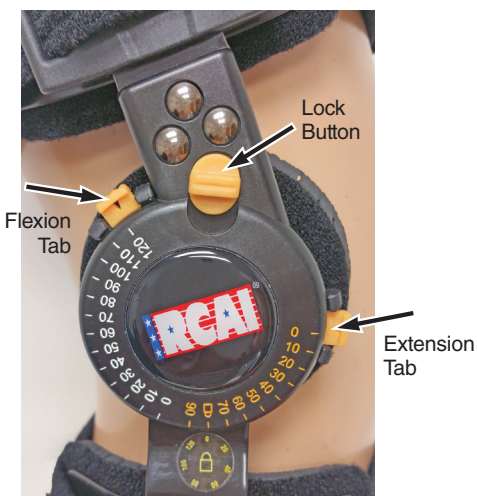

Setting the hinge

1. Push in flexion and extension tabs at the sides of the hinge and slide to desired degree on angles.
2. Lock range of motion setting by sliding down the lock button at the top of the hinge.

SIZING

Size	Overall Length	Max Thigh Circ.	Model #
One Size	18 1/2 – 22" (47.0 - 55.9 cm)	29" (73.7 cm)	95ROM-KO

Limited Warranty Coverage: Any defect in material or workmanship solely limited to the Kydex® plastic frame or metal portion or hinge mechanisms for 1 (one) year from date of purchase.